

EyeBrid™

Silicone Multifocal


Silicone Hydrogel hybrid contact lens

With EyeBrid Silicone Multifocal

BC = K
 P = Spectacle refraction + vertex distance
 Add: Add spectacles Design: D lens on preferred eye
 N lens on other eye

Evaluate

1. Control visual acuity with the lens as a soft multifocal lens for D and N vision
2. Do a slit lamp exam and evaluate as a soft lens
 If satisfactory, leave the patient with the lenses
 If centration and/or mobility are unsatisfactory, change the skirt

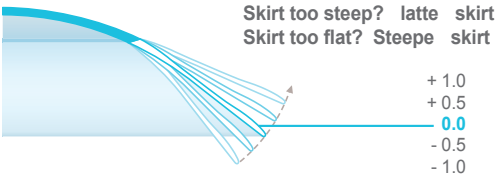

Too flat	Optimal	Too steep
Steepe BC mm	res ri e	latte BC mm

in o eyebridsilicone.com

