Ophtecs

ENGLISH | Instructions for Use

Disinfecting, cleaning, rinsing and storing system for rigid gas permeable contact lenses.

FOR ALL RIGID GAS PERMEABLE CONTACT LENSES


Wash your hands well with mild soap before handling your contact lenses.

1 DISINFECTING, CLEANING AND NEUTRALIZING

- Place the lenses into the case's lens holders.
- Close the case lid on one side, put the neutralizing and cleaning tablet in, and fill up to 90% of the case with the orange disinfecting solution.
- Close the lid on the other side tightly.
- Soak the lenses for over four hours (or overnight).
- Confirm that the lenses and the solution are clear.

2 RINSING LENSES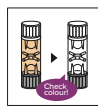

- Open the lid on one side and discard the solution in the lens case.
- Pick up the cleaned/disinfected lenses from the holder. Hold them with your fingers and rinse both sides with the rinsing solution for at least 10 seconds.
- Always use the **cledeu CareSolution** to rinse the lenses.
- Avoid using saline or MPS solutions containing sodium chloride as a preservative or disinfectant.

3 WASHING LENS CASE

- After the care of the lenses is completed, wash the lens case with **cledeu CareSolution** and allow it to air dry.

STORING LENSES

- Store the lenses at room temperature after disinfecting, cleaning, and neutralizing with **cledeu GP**. Do not open the case until using the lenses.
- To store the lenses for 30days or longer, disinfect and clean with **cledeu GP** in accordance with disinfecting, cleaning and neutralizing procedure as above every 30 days.
- Rinse the lenses with **cledeu CareSolution** before wearing them in accordance with the rinsing lens procedure.

PRECAUTIONS

- To avoid serious eye infections or damage, always follow the user instructions.
- Use the disinfecting solution and the neutralizing and cleaning tablet together.
- Close the bottle tightly after use.
- Always use the **cledeu GP** system components including the lens case and **cledeu CareSolution**.
- Discard any remaining solution 90 days after the expiration date of the solution.
- Do not use for soft contact lenses.
- Do not use the **cledeu CareSolution** to disinfect or store lenses.
- Do not touch the tip of the bottle to avoid contamination.
- Do not swallow the solution or tablet. If you do so, drink a large amount of drinking water immediately.
- Do not be used in the eye. If you do so, immediately rinse the eye with sterile saline solution or eye washes.
- Do not use the tablet if it is damaged or discoloured.
- Do not use the solution. (The solution after use is neutralized and cannot disinfect the lens. To prevent eye infection, do not reuse the solution.)
- Do not use the product after the expiry date.
- Do not use the product if the bottle or package is soiled or damaged.
- Do not use the solution if it is cloudy or discoloured.
- Do not wear lenses without rinsing them with **cledeu CareSolution** after disinfecting, cleaning and neutralizing.
- Do not use the product together with another solution, tablet, or pharmaceutical drug.
- Do not use tap water to rinse the lenses or lens case.
- Keep out of reach of children.
- Do not transfer the solution to another bottle or case.
- To prevent contamination, use a new lens case with each purchase.
- Use the tablets immediately after opening the package.
- After using up all of the neutralizing and cleaning tablets, discard any remaining solution in the bottle.
- Wash the solution adheres to your clothes, wash with water immediately.
- Rinse off any remaining solution on the wash stand with water after each use to prevent staining on the surface of the wash stand.
- Store products in a dry place at room temperature, avoiding direct sunlight.
- If you experience such symptoms as eye pain and irritation, remove your lenses immediately and consult your eye doctor or an eye care professional.
- Do not use the product if you are allergic to any of the ingredients used in it.
- Do not change the contact lens care system without consulting your eye care professional.
- If you run out of the **cledeu CareSolution**, please purchase additional units of the **cledeu CareSolution**.
- Do not use as a topic to reproduction and can harm babies if taken by pregnant or lactating women. Although this product also contains borate, the amount of borate acid in this product is at a concentration that does not pose a health hazard. This product is safe for use by children and pregnant women, as long as the product's directions are followed. Although this chemicals differs from person to person. For sensitive individuals, the recommendation is to rinse the lens with sterile saline solution before wearing.

INGREDIENTS

[Disinfecting solution] Povidone-iodine (0.05%) / Boric acid **STERILE A**
 [Neutralizing and cleaning tablet] Sodium sulfite (2.4 mg/tablet), Proteolytic enzyme (8.0 mg/tablet)
 [Rinsing solution] Hydrogen peroxide as preservative (0.004%) / Boric acid **STERILE A**

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