EN Cleaner

For cleaning.

For all types of contact lenses.

Composition:

PHMB 0,002%, METHYLOXIRAN/ETHYLENOXID POLYMER, COCOAMPHODIPROPIONATE, PROPYLENE GLYCOL, HYDROGEN CHLORIDE, PURIFIED WATER.

Precautions and instructions for use

The non-sterile cleaner should be used in combination with a sterile multipurpose contact lens or rinsing solution.

Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel.

Do not use when tamper-evident seal is broken before the first time use. Unscrewing the cap from the bottle will break the tamper-evident seal.

Shake the bottle before use.

Immediately after removal of the lenses, clean them with a few drops of cleaner.

Rub the lens gently between the palm of your hand and your finger during 30 seconds.

After cleaning, rinse both sides of the lens with a sterile multipurpose contact lens or rinsing solution.

After rinsing, the lenses can be inserted.

Close the bottle tightly immediately after use.

Storage conditions

Use within 3 months after opening.

Keep out of reach of children. (See image 1)

Warnings and contra-indications

Do not insert lenses directly into the eye after using the cleaner without rinsing the lens with a sterile contact lens care solution.

Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor.

If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lens solution.

Do not use when the packaging is damaged. (See image 2)

Do not use after the expiry date.

Do not change your contact lens care system without consulting your contact lens specialist.

Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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