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gb SOLUTION FOR RIGID AND GAS-PERMEABLE CONTACT LENSES

• cleans • disinfects • stores • rinses • wets

INSTRUCTIONS FOR USE:

1. Before handling your lenses, wash your hands with soap and water.
2. Place your lens on the palm of your hand and rub it carefully with a few drops of LCS GP MULTI, on both sides for a few seconds.
3. Rinse your lenses with LCS GP MULTI.
4. Place your lenses in the lens case, and be sure to cover these with LCS GP MULTI.
5. Keep lenses at least four hours in LCS GP MULTI (or overnight) to grant an adequate disinfection.
6. Once the disinfection process is done, you may insert your lenses directly to your eye or you may rinse them previously with LCS GP MULTI.
7. Keep your lens case clean and be sure to replace the LCS GP MULTI in your lens case daily.

Last revision date: 2009/11

PRECAUTIONS:

Once opened, the bottle should not be used after 90 days.
Maintain the bottle well closed after use.
Do not use the product if the safety seal is missing or broken.
You may store your lenses up to 30 days in the closed lenscase.
Do not ingest the solution and Keep away from childrens reach. Keep at room temperature.
Do not allow the dropper to come into contact with any surface to avoid contamination.
In case of irritation, consult your contact lens specialist.
Before using any ophtalmic medicine, inform your doctor you are a contact lens user.
Do not use if you are allergic to any of the components.
Never use tap water to rinse your lenses or lens case.
Do not use after the expiration date.
Discard the solution in the lenscase after each use.

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