#### SIN CONSERVANTES

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#### PRESERVATIVE-FREE

**OPHTHALMIC SOLUTION FOR OCULAR HYGIENE, EYEWASH AND CONTACT LENS RINSE.** Physiologic pH, suitable for eye surface.

#### Instructions for use:

- Before handling your lenses, wash your hands thoroughly with soap and water.
  - Take the vial of LCS Saline Unidose and open by pulling on the upper tab.
  - Once opened, use the vial immediately.
    - For contactlense use: Rinsing.
 Alter the cleaning process and whenever necessary, place the lens on the palm of your hand and rinse thoroughly with abundant solution. Gently rub lens between your fingertips. After rinsing, you can place the lens directly in your eye.
    - For ophthalmic use: Cleaning, instilling or as an eyewash.
 Place a few drops of LCS Saline Unidose in the eye to remove irritating substances or particles, or apply to cotton to gently clean the eyelid and the area around the eyes. Use a clean cotton for each eye.
    - Use with enzymatic tablets for contact lens cleaning: Disolution.
 Use the solution according to the instructions for use of the tablets.
  - After one use, discard the vial and any leftover solution to avoid contamination.
- Last revision date: 2009/11

#### Precautions:

- For external use only. Do not inject.
- Once the vial is open, use immediately. This product does not contain preservatives and after use it must therefore be discarded to prevent a microbial contamination.
- Do not ingest the solution and keep out of children's reach.
- Do not use the vial if damaged.
- In case of eye irritation, consult your eyecare specialist.
- Store at room temperature.
- Do not use after expiry date.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.
- Do not use if you are allergic to any of the components.

#### Composition:

Sterile, buffered, isotonic and aqueous solution.

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AVIZOR S.A.