
Disinfecting, cleaning, rinsing and storing system for rigid gas permeable contact lenses.
FOR SCLERAL CONTACT LENSES
Wash your hands well with mild soap before handling your contact lenses.

1 DISINFECTING, CLEANING AND NEUTRALIZING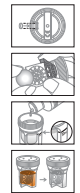

- Set a neutralizing and cleaning tablet at the specified position (sit) of the dedicated case body. Be sure to use the dedicated case. ⚠️Setting the tablet at a position other than the specified position may lead to unsuccessful neutralisation, resulting in colouration of the lens.
- Place the lenses in the lens holder of the case.
- Fill the case with the disinfecting solution up to the marked line.
- Close the case lid and let it stand for at least four hours.
- Confirm that the lenses and the solution are clear.

2 RINSING LENSES

Pick up the cleaned/disinfected lenses from the holder. Hold them with your fingers and rinse both sides with the rinsing solution for at least 10 seconds.
⚠️Always use the **cleadew CareSolution** to rinse the lenses.
Avoid using saline or MPS solutions containing sodium chlorite as a preservative or disinfectant.

STORING LENSES

- To store the lenses, be sure to follow the instructions of eye care professionals.
- When planning to wear lenses that have been stored for 24 hours or longer, it is recommended to start care from the beginning before wearing them.

⚠️ PRECAUTIONS

- ❗ To avoid serious eye infections or damage, always follow the user instructions.

❗ Use the disinfecting solution and the neutralizing and cleaning tablet together.

❗ Close the bottle tightly after use.

❗ Always use the **cleadew SL** system components including the lens case and **cleadew CareSolution**.

❗ Discard any remaining solution 90 days after opening the bottle.

❗ Do not use for soft contact lenses.

❗ Do not use the **cleadew CareSolution** to disinfect or store lenses.

❗ Do not touch the tip of the bottle to avoid contamination.

❗ Do not swallow the solution or tablet. If you do so, drink a large amount of drinking water immediately.

❗ Not to be used in the eye. If you do so, immediately rinse the eye with sterile saline solution or eye washes.

❗ Do not use the tablet if it is damaged or discoloured.

❗ Do not reuse the solution. (The solution after lens care is neutralized and cannot disinfect the lens. To prevent eye infection, do not reuse the solution.)

❗ Do not use the product after the expiry date.

❗ Do not use the product if the bottle or the package is soiled or damaged.

❗ Do not use the solution if it is cloudy or discolored.

❗ Do not wear lenses without rinsing them with **cleadew CareSolution** after disinfecting, cleaning and neutralizing.

❗ Do not use the product together with another solution, tablet, or pharmaceutical drug.

❗ Do not use tap water to rinse the lenses or lens case.

❗ Keep out of reach of children.

❗ To prevent bacterial contamination, do not transfer the disinfecting solution to another container, nor replenish it with water or other liquid medical agent.

❗ To prevent contamination, use a new lens case with each purchase.

❗ Use the tablets immediately after opening the package.

❗ After using up all of the neutralizing and cleaning tablets, discard any remaining solution in the bottle.

❗ If the solution adheres to your clothes, wash with water immediately.

❗ Rinse off any remaining solution on the wash stand with water after each use to prevent staining on the surface of the wash stand.

❗ Store products in a dry place at room temperature, avoiding direct sunlight.

❗ If you experience such symptoms as eye pain and irritation, remove your lenses immediately and consult your eye doctor or an eye care practitioner.

❗ Do not use the product if you are allergic to any of the ingredients used in it.

❗ Do not change the contact lens care system without consulting your eye care professional.

❗ In the unlikely event that the lens is coloured after care, do not wear the coloured lens.

❗ If you run out of the **cleadew CareSolution**, please purchase additional units of the **cleadew CareSolution**.

❗ Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women. Be careful and avoid direct use of the solution in the eye; do not swallow. Proper use can reduce the risk; therefore, this product can be used by children or pregnant or lactating women. Sensitivity to chemicals differs from person to person. We recommend rinsing the lenses with boric acid-free sterile physiological saline before wearing them if you are susceptible.

INGREDIENTS

[Disinfecting solution] Povidone-iodine (0.05%) ⚠️Acido borico **STERILE A** 
[Neutralizing and cleaning tablet] Sodium sulfite (2.4 mg/tablet),
Proteolytic enzyme (8.0 mg/tablet)
[Rinsing solution] Hydrogen peroxide as preservative (0.004%) ⚠️Acido borico **STERILE A** 

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OPHTECs Europe B.V.