ENGLISH | Instructions for Use

Disinfecting, cleaning and storing system for soft contact lenses.
FOR ALL SOFT CONTACT LENSES INCLUDING SILICONE HYDROGEL LENSES

Read this instruction sheet carefully before using cleadew, and keep it for future reference. To avoid eye infections or damage, always follow the user instructions.

COMPOSITION AND DESCRIPTION

Do not use cleadew if allergic to any of the ingredients used in cleadew.

Diagram showing the components of the cleadew system: lens case, disinfecting, neutralizing, and cleaning tablets, and a solution bottle. Includes instructions on how to use the lens case and the solution.

USER INSTRUCTIONS

Step-by-step visual instructions for using cleadew: 1. Open lens holders and place lenses. 2. Place a tablet of cleadew in the lens case. 3. Soak lenses for at least 4 hours. 4. Check colour. 5. Remove lenses. 6. Rinse lenses in the palm of your hand.

STORAGE

After completing the above steps, in case of storing the lens for more than 24 hours, you should replace the solution with cleadew. You may store the lens in the unopened lens case up to 7 days, and then repeat the care steps from the beginning before wearing your lens.

WARNINGS

- Do not use cleadew if allergic to any of the ingredients used in cleadew.
If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
Do not use cleadew with any eye medication or other contact lens care products.

PRECAUTIONS

- Read this instruction sheet carefully before using cleadew and keep it for future reference.
Do not use the solution. The solution after lens care is neutralized and does not contain any disinfectant.
If the solution does not turn orange after you have added a tablet, discard the solution, wash and dry the lens case, and then repeat the care steps from the beginning.
Ensure that solution and lens are completely clear before wearing the lens.
Always use the specially designed lens case provided with cleadew.
Discard any remaining solution 3 months after opening the bottle.
Always use cleadew A and cleadew B together.
Do not swallow the solution or the tablets.
Do not touch the tip of cleadew B to avoid contamination.
Do not use cleadew for hot disinfection.
Do not use tablet if broken.
Do not use cleadew after the expiry date.
Do not use the solution or tablet if it is discoloured.

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Diagram showing the components of the cleadew system in French: lens case, disinfecting, neutralizing, and cleaning tablets, and a solution bottle. Includes instructions on how to use the lens case and the solution.

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Diagram showing the components of the cleadew system in German: lens case, disinfecting, neutralizing, and cleaning tablets, and a solution bottle. Includes instructions on how to use the lens case and the solution.

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Diagram showing the components of the cleadew system in Spanish: lens case, disinfecting, neutralizing, and cleaning tablets, and a solution bottle. Includes instructions on how to use the lens case and the solution.

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Diagram showing the components of the cleadew system in Italian: lens case, disinfecting, neutralizing, and cleaning tablets, and a solution bottle. Includes instructions on how to use the lens case and the solution.

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