F LCS ENZYM

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Subtilisin A

GB LCS ENZYM

ENZYMATIC CLEANING TABLETS TO REMOVE DEPOSITS FROM SOFT AND GAS PERMEABLE CONTACT LENSES.

INSTRUCTIONS FOR USE:

1. Before handling your lenses, wash your hands.
2. Fill the lens case with solution **Unica Avizor**, solution **LCS Saline**, solution **Avizor Gp Conditioner** or solution **LCS GP**.
3. Place an **LCS Enzym** into the lens case, filled with solution, insert both contact lenses into their respective lens holders and screw on the lid.
4. Maintain the lenses submerged inside the lens case for at least 2 hours, and not more than 24 hours.
5. Remove lenses from lens case and rinse them thoroughly with solution **Unica Avizor**, solution **LCS Saline**, solution **Avizor Gp Conditioner** or solution **LCS GP**.
6. Repeat this cleaning process every week.

Last revision date: 2009/11

PRECAUTIONS:

- After enzymatic cleaning, rinse the lenses thoroughly before inserting. Enzymes could irritate your eyes, and must be removed from lenses by intensive rinsing.
- Keep tablets in a dry and fresh place, at room temperature.
- Do not ingest the tablets and keep out of children's reach.
- In case of irritation, consult your contact lens specialist.
- Do not use after the expiration date.
- Discard the solution in the lens case after each use.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.

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